GERMAN  
POOL

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All-Round Electronic Body  
Analyser Scale  
全能人體分析電子磅

# ESB-315

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Please read these instructions and warranty information carefully before use and keep them handy for future reference.

使用說明書 OPERATING INSTRUCTIONS

# 目錄 Table of Contents

---

注意事項 Notice Before Use 03

產品結構 Product Structure 04

操作說明 Operating Instructions 06

常見故障及處理 Troubleshooting 11

清潔與保養 Care & Maintenance 13

技術規格 Technical Specifications 13

保用條款 Warranty Terms & Conditions 15

# 注意事項 Notice Before Use

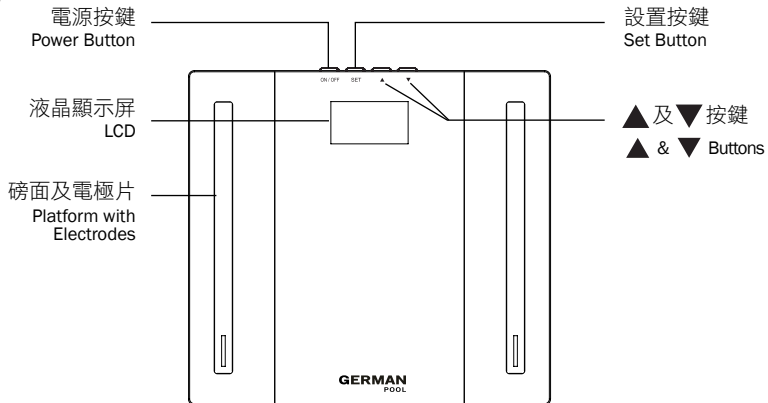
---

1. 在使用本機前，請仔細閱讀本說明書以熟悉本產品的功能和操作方法。
  2. 本產品不適用於體內裝置金屬儀器者，如起搏器、節育環、金屬片及螺絲等；否則將有可能導致儀器損壞或量度不準確。如有疑問，請諮詢您的醫生。
  3. 本產品透過BIA (BIOELECTRICAL IMPEDANCE ANALYSIS) 技術來量度體重，阻抗及評估身體脂肪及水分、肌肉及骨頭百分比。本產品適合家庭使用於10-17歲的健康兒童，以及生活方式活躍、一般活躍和不活躍的健康成人。
  4. 本產品適用於10-85歲人士。
  5. 請使用規定的電池（請參照“操作說明”部分）。
  6. 運動員模式適用範圍為15-85歲。
  7. 脂肪比率會隨著體內水分而改變，缺水狀態或者溢水狀態都會影響人體內的脂肪比，如喝酒、月經期、生病、劇烈運動等。
  8. 本產品不適用於孕婦，否則將導致測量結果不準確，且對胎兒的影響尚未可知。
  9. 使用脂肪水分度量時，請赤腳進行。
  10. 本產品只供家庭測量體重，不可用於商業或其他用途。
  11. 請勿拆卸本機，不恰當的操作會導致損傷。
  12. 當您的身體或腳上帶有很多水時，特別是剛剛沐浴後請勿稱量，謹防滑到。
  13. 浮腫、發燒、骨質疏鬆、接受洗腎療程等病人不可使用本產品。
  14. 服用血管擴張或收縮藥物或其他心血管有關藥物的人士不可使用本產品。
  15. 電池有造成窒息的危險，請勿讓兒童接觸。
1. Please read this section carefully to familiarize yourself with features and operations before using the unit.
  2. Do not use the scale on people who have body implants such as a pacemaker, artificial limbs, contraceptive devices, or metal plates or screws. It may cause the device to malfunction or produce an inaccurate result. When in doubt, consult your medical officer.
  3. This scale is intended to measure body weight and impedance and estimate percentage of body fat and body water, muscle mass and bone mass using BIA (Bioelectrical Impedance Analysis). It is intended for use by healthy children 10-17 years old and healthy adults with active, moderately active, to inactive lifestyles for body composition assessment in the home environment.
  4. This product is intended for adults and children (ages 10 to 85).
  5. Make sure to use only the type of battery stated (see Section "Operating Instructions").
  6. The "Athlete" mode is applicable only to people 15-85 years of age.
  7. Body fat percentage estimates will vary with the amount of water in the body, and can be affected by dehydration or over-hydration due to such factors as alcohol consumption, menstruation, illness, intense exercise, etc.
  8. Do not use on pregnant women. The result is inaccurate and effects on the fetus are unknown.
  9. For body fat and body water estimates, always estimate in bare feet.
  10. Do not disassemble the scale as incorrect handling may cause injury.
  11. Do not step on the scale when your body or feet are wet, especially after bathing or showering to prevent slipping.
  12. This scale is only used in household for body weight measurement, and not for commercial or other use.
  13. Edema, fever, osteoporosis and patients receiving dialysis treatment should not use this product.
  14. People taking blood vessel dilation or contraction drugs or other cardiovascular-related drugs should not use this product.
  15. Batteries may pose a choking hazard. DO NOT let children handle batteries.

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### 1. FUNCTION KEYS

#### ON/OFF

- Turn the scale on or off
- Enter body fat and total body water estimation mode

#### SET

- Confirmation button for selection

#### ▲ or ▼

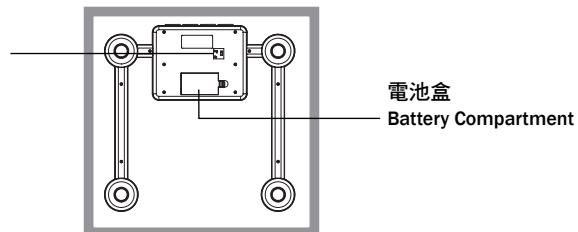
- Changes value of height and age, toggle between Male and Female, toggle between Normal and Athlete mode in Body Fat function
- Select memory 1-12 in Memory mode & Recall mode
- Recall settings of memory locations

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
**Slide Switch**  
Adjust the conversion switch  
located at the bottom of the  
scale to change the measuring  
unit between kg/cm, lb/in. User  
selectable unit: kg/cm or lb/in.

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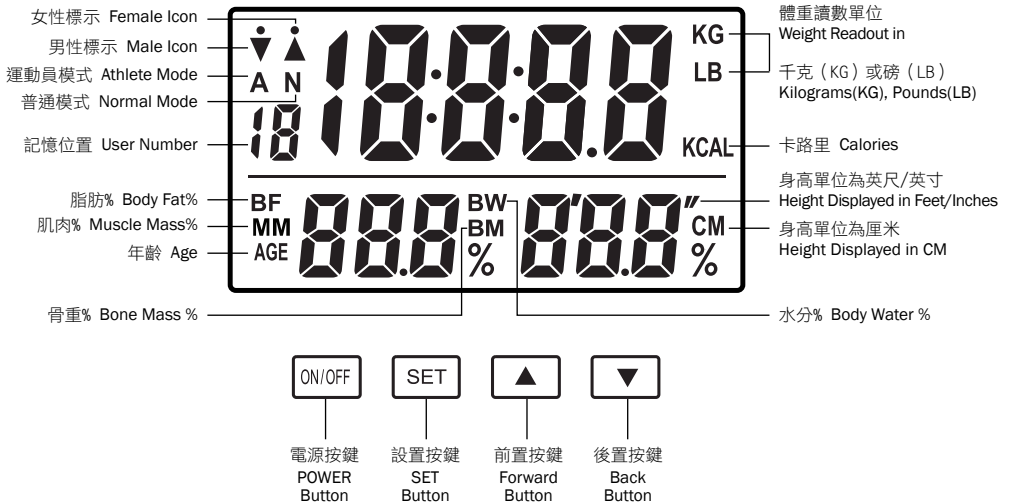
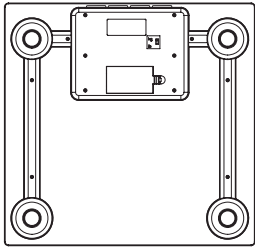
開啟電池門 Open the battery door

## 1. Insert the battery

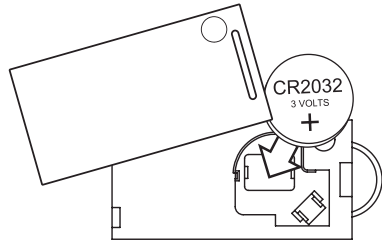
This scale operates on 1 x CR2032 lithium battery (included). Open the battery cover on the back of the scale unit. Remove any plastic strip before first use or insert a new battery for replacement. Be sure the polarity of the battery is set correctly for the scale to function properly. If you do not intend to use this unit for a prolonged period of time, it is advisable to remove the battery before storing.

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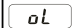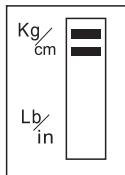
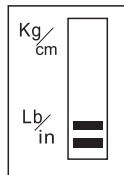
Lb (磅)

## 2. Set Unit

This scale is set to pounds. You can toggle the unit of measurement switch located on the back of the scale for kilograms or pounds.

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3. Some models have a static cling label on the LCD display to prevent scratching. Remove before use.

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## 4. Set the right position

Always use the scale on a flat and hard floor surface. Do not use on carpets.

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- 請勿濕腳磅重或踩踏磅秤邊緣，謹防滑倒。
- To avoid injury, do not step with wet feet or on the edge of the platform. Beware of slipping

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## 5. Start-up the Scale

Press lightly on the scale platform to start-up the scale, the display shows “0000” and then turns off. The scale is ready to use.

### ⚠ 注意 PRECAUTION !

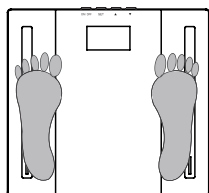
- 在插入電池/更換電池或每次移動或磅秤被碰撞後，磅秤必須重新啟動。
- The scale must be started up after battery insertion/replacement of battery or each time it is moved or bumped.

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
## 6. Operating Guidelines

The accuracy of the results depends on how you stand on the scale. Position your feet with maximum contact on the metal electrodes on the platform. This ensures the best contact between your feet and metal contacts. Stay on the scale until the body fat estimation is completed and the result is displayed. Clean, slightly moist feet will provide the best results.

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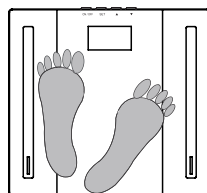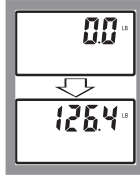
圖2 (Fig.2)

## 7. Weighing-only Operation

- Step onto the scale and stand still while the weight is being measured (Fig. 1).
- The screen displays the weight (Fig. 2).
- The scale turns off automatically after use.

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## 8. Body Fat and Total Body Water Estimating Operation with User Memory

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- A few steps must be followed before estimating body fat and total body water.

#### PROGRAM PERSONAL DATA

- Input the user's gender, age and height. Once the information is memorized, it will only need to be reentered if there is a change to the data.

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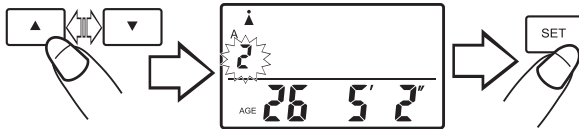
### i. Press the ON/OFF key to turn the scale on.

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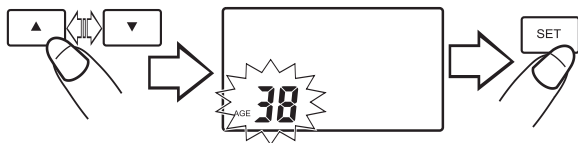
## iv. Set Age

Press the [▲] or [▼] key to adjust the age.  
Press [SET] to confirm.

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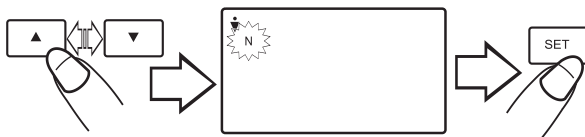
## v. Set Gender

Press the [▲] or [▼] key to toggle  
between the male (♂) & female (♀) icons.  
Press [SET] to confirm.

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
## vi. Select Normal or Athlete Mode

Press the [▲] or [▼] key to toggle  
or athlete. Press [SET] to confirm all settings.

NOTE: Athlete mode is only available for those  
15-85 years of age. For those aged 14 or  
under, this step is automatically by-passed.

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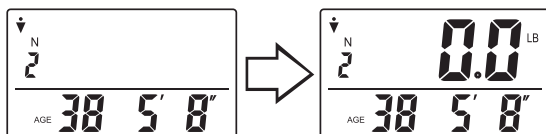vii. All personal information stored to memory  
will be displayed. The scale will automatically  
shut off after a few seconds. Your personal  
information is saved in memory.

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viii. Repeat step (i)-(vii) to set other pre-programmed  
user numbers.

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9. The scale features 12 personal memory settings. This allows users to store and recall their own height, age, gender and normal/athlete mode selection. Follow steps in the previous section “Program Personal Data” to input your personal information into a memory location.

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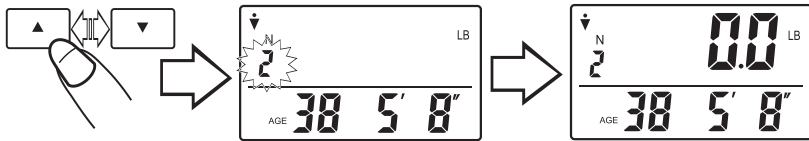
## A. To recall Settings from Memory for Body Fat and Total Body Water Estimation

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You must be in bare feet for body fat and total body water estimations. Remove your shoes and socks before proceeding. Clean, slightly moist feet will provide the best results. Position your feet for maximum contact with the metal electrodes on the platform.

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
1. Press the ON/OFF key to turn the scale on.
2. Press the 【▲】 or 【▼】 key to toggle between memory location (1-12). The screen will show the previous reading, then “0.0”.  
(NOTE: The previous reading for that user number will appear after first use.)

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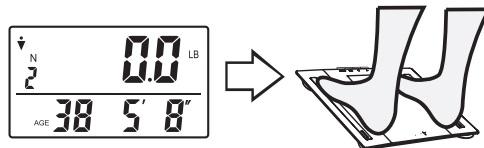3. When “0.0” appears on screen, step on the scale with bare feet, positioning your feet evenly on the scale platform with maximum contact with the metal electrodes.

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
4. Stand still while the scale measures your weight and estimates your body composition.

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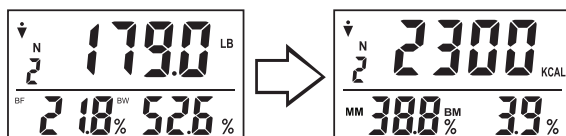5. Your weight, body fat % (BF) then total body water % (BW) are displayed. Next your Calories (KCAL), muscle mass % (MM) and bone mass % (BM) are displayed sequentially twice.

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## B. To Turn Off the Scale

To turn the scale off, press the ON/OFF key while programming data. Or, if no key has been pressed for a few seconds, the scale turns off automatically.

# 常見故障及處理 Troubleshooting

無重量顯示 No weight is displayed?	請檢查電池是否有電。如果不是，請參照“操作說明”部分。 Check if the scale is powered up and started up. If not, please refer to the Section “Operating Instructions”.
螢幕顯示 “Lo” The display reads “Lo”	低電顯示，請更換電池。 Battery is running low. Replace the battery.
螢幕顯示 “Err0” The display reads “Err 0”	歸零異常。從磅上下來，待自動關機後，請重新輕按磅面進行初始化，螢幕顯示 “0000” 後自動關機，初始化完成。 Initialization error. Step off the scale and wait until the scale is automatically switches off. Start the scale again by pressing lightly on the scale platform to re-initialize the scale. The scale is ready for use again.
螢幕顯示 “Err1” The display reads “Err 1”	不穩定異常。從磅上下來，待自動關機後，重新站在磅上進行稱重，並注意在稱重過程中保持靜止不動。 Instability error. Step off the scale and wait until the scale automatically switches off. Step on the scale to repeat the measurement again, standing still while computation is in progress.
稱重過程中，螢幕顯示 “Err2” The display reads “Err 2” when measuring	超重預警。立即從磅上下來，否則將對磅秤造成永久性傷害。 Overload warning. Remove the weight immediately; otherwise, permanent damage to the scale will occur.

# 常見故障及處理 Troubleshooting

<p>脂肪及水分測量過程中，螢幕顯示“Err3” The display reads “Err 3” when estimating body fat and body water values</p>	<p>測量異常。電阻無法測量到。請確保稱重過程中保持靜止並使腳底最大面積接觸電極片。如若仍然不能測量到，請參考“操作說明”部分。您也可以通過微微濕潤你的腳底，促進電極接觸。 Measurement error. Impedance cannot be measured. Please make sure that you are standing still on the scale and maintaining maximum contact between your feet and the metal contacts. If not, please refer to “Operating Instructions” section. You may need to moisten your feet to improve the electrical contact.</p>
<p>Err1至Err3 都試過了，可仍不能解決問題 After I tried the corrective actions for Err 1 to Err 3, I still can't solve the problem</p>	<p>解決方案都試過了，可問題仍然存在，請移除電池一分鐘後重新安裝。 If Err 1 to Err 3 persists after following the corrective actions, remove and reinsert battery after 1 minute.</p>
<p>我已經試過所有的解決方案了，仍然不能解決問題 I have tried all corrective actions, but still can't solve the problem</p>	<p>請聯繫德國寶維修中心或授權經銷商。 Please contact German Pool Repair Centre or an authorized dealer.</p>
<p>為什麼我使用不同品牌的磅秤，測量的脂肪結果不同？ Why do I get a different body fat reading when I use a different brand of body fat scale?</p>	<p>不同的脂肪磅秤運用不同的數學原理測量身體不同部分獲得脂肪百分比。我們建議您無需進行不同產品的比較，只需每次都使用同一款產品進行測量跟蹤。 Different body fat scales take estimations around different parts of the body and use different mathematic algorithm to estimate the percentage of the body fat. The best advice is not to make comparison from one device to another, but to use the same device each time to monitor any change.</p>

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產品規格及設計如有變更，恕不另行通知。  
Specifications are subject to change without prior notice.

中英文版本如有出入，一概以中文版為準。  
If there is any inconsistency or ambiguity between the English version and the Chinese version, the Chinese version shall prevail.

本產品使用說明書以 [www.germanpool.com](http://www.germanpool.com) 網上版為最新版本。  
Refer to [www.germanpool.com](http://www.germanpool.com) for the most up-to-date version of the Operating Instructions.

請填妥以下空格並與發票一併保存，以便維修時核對資料。用戶提供本公司之所有資料必須正確無誤，否則免費保用將會無效。  
Please fill out the form below. This information and the purchase invoice will be required for any repairs. Warranty will be invalid if information provided is found to be inaccurate.

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## Warranty Terms & Conditions

Under normal operations, the product will receive a 1-year full warranty provided by German Pool (effective starting from the date of purchase).

\* This warranty is not valid until Customer's registration information is returned to and received by our Service Centre within 10 days of purchase via one of the following methods:

- 1) Visit our website and register online: [www.germanpool.com/warranty](http://www.germanpool.com/warranty).
- 2) Complete all fields on the attached Warranty Card and mail it back to our Customer Service Centre along with a photocopy of the purchase invoice (Warranty Card and invoice copy will not be returned to Customer).

1. Customer who fails to present original purchase invoice will be disqualified for free warranty.
2. Customer should always follow the operating instructions. This warranty does not apply to:
  - labour costs for on-site installation, check-up, repair, replacement of parts and other transportation costs;
  - damages caused by accidents of any kind (including transport and others);
  - plastic parts and accessories;
  - operating failures resulted from incorrect voltage, improper operations and unauthorized installations or repairs.
3. This warranty is invalid if:
  - purchase invoice is modified by unauthorized party;
  - product is used for any commercial or industrial applications;
  - product is repaired or modified by unauthorized personnel, or unauthorized parts are installed;
  - serial number is modified, damaged or removed from the product.
4. German Pool will, in its discretion, repair or replace any defective part.
5. This warranty will be void if there is any transfer of ownership from the original purchaser.

## Repair Centre - Small Appliances

German Pool (Hong Kong) Limited  
8/F, China Travel (Cargo) Logistics Centre  
1 Cheong Tung Road  
Hung Hom, Kowloon, Hong Kong  
Tel : +852 2333 6249  
Fax : +852 2355 7100  
Email : repairs@germanpool.com

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This warranty is valid only in Hong Kong

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© All rights reserved. Copying, reproducing or using the contents of this manual is not allowed without prior authorization from German Pool, violators will be prosecuted.

[www.germanpool.com](http://www.germanpool.com)